


# QUICK-SIL™

50ml. Working Instructions

## STEP ONE



Prepare Area

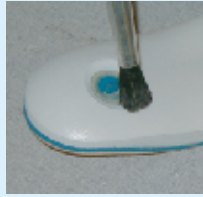

Brush Off  
Dust

## STEP TWO\*



Shake Well



Equal  
Amounts



Mix



Spread with  
Cotton Swab

To adhere Quick-Sil to material, spread a small amount of QuickPrime over area. (\*Not necessary if using silicone adhesive)

## STEP THREE



Set Up  
Mixing Gun



Add Static  
Mixer



Inject  
Quick-Sil



Wait Five  
Minutes

Follow the steps below to adhere Quick-Sil to materials using silicone adhesive:




Remove Set  
Silicone



Apply thin coat  
of silicone adhesive



Place Quick-Sil

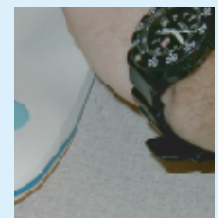

Wait at least  
30 minutes